

Sterilisation and Instructions for Use – Product Schedule

Prophy Polishing Brushes, RA and Screw

Classification: Class 2a Medical Device – (single use device)

GMDN Code	GMDN Term	Basic UDI-DI	Medical Device	MD Class
64015	Dental Professional Prophylaxis Brush, single use	5051717JC01T5	JC, Junior Cup Brushes	Ila
64015	Dental Professional Prophylaxis Brush, single use	5051717PB01UA	KS, KSO, MT Brushes	Ila
64015	Dental Professional Prophylaxis Brush, single use	5051717WH01WR	W,WGH Brushes	Ila

JC/WBRA	Junior Cup, White Bristle, RA
JC/WBSC	Junior Cup, White Bristle, Screw
JC/WNRA	Junior Cup, White Nylon, RA
JC/WNSC	Junior Cup White Nylon, Screw
JC/PKNRA	Junior Cup, Pink Nylon, RA - Soft
JC/PUNRA	Junior Cup, Purple Nylon, RA - Medium
JC/PBNRA	Junior Cup, Blue Nylon, RA - Hard
JC/PKNSC	Junior Cup, Pink Nylon, Screw - Soft
JC/PUNSC	Junior Cup, Purple Nylon, Screw - Medium
JC/PBNSC	Junior Cup, Blue Nylon, Screw - Hard
JC/WNGRA	Junior Cup, White Nylon, Extra Soft (Gold Cup), RA
JC/WNGSC	Junior Cup, White Nylon, Extra Soft (Gold Cup), Screw
JC/ORNRA	Junior Cup, Orange Nylon, RA
JC/YNRA	Junior Cup, Yellow Nylon, RA
JC/SDRA	Junior Cup, Stain Devil, RA
JC/WFRA	Junior Cup, Whitening, RA
JP/WNRA	Junior Polymer Cup, White Nylon, RA
JP/PKNRA	Junior Polymer Cup, Pink Nylon, RA - Soft
JP/PUNRA	Junior Polymer Cup, Purple Nylon, RA - Medium
JP/PBNRA	Junior Polymer Cup, Blue Nylon RA - Hard
W/WGHRA	Wheel, White Goat Hair, 15mm, RA

KS/WBRA001	Pencil Brush, One Knot, White Bristle, RA
KS/WBRA002	Pencil Brush, One Knot White Bristle, Pointed, RA
KS/WBSC001	Pencil Brush, One Knot, White Bristle, Screw
KS/WBSC002	Pencil Brush, One Knot, White Bristle, Pointed, Screw
KS/WNRA001	Pencil Brush, One Knot, White Nylon, RA
KS/WNRA002	Pencil Brush, One Knot, White Nylon, Pointed, RA
KS/WNSC001	Pencil Brush, One Knot, White Nylon, Screw
KS/WNSC002	Pencil Brush, One Knot, White Nylon, Pointed, Screw
KS/PKNRA001	Pencil Brush, One Knot, Pink Nylon, RA - Soft
KS/PKNRA002	Pencil Brush, One Knot, Pink Nylon, Pointed, RA - Soft
KS/PUNRA001	Pencil Brush, One Knot, Purple Nylon, RA - Medium
KS/PUNRA002	Pencil Brush, One Knot, Purple Nylon, Pointed, RA - Medium
KS/PBNRA001	Pencil Brush, One Knot, Blue Nylon, RA - Hard
KS/PBNRA002	Pencil Brush, One Knot, Blue Nylon, Pointed, RA - Hard
KS/PKNSC001	Pencil Brush, One Knot, Pink Nylon, Screw - Soft
KS/PKNSC002	Pencil Brush, One Knot, Pink Nylon, Pointed, Screw - Soft
KS/PUNSC001	Pencil Brush, One Knot, Purple Nylon, Screw - Medium
KS/PUNSC002	Pencil Brush, One Knot, Purple Nylon, Pointed, Screw - Medium
KS/PBNSC001	Pencil Brush, One Knot, Blue Nylon, Screw - Hard

Sterilisation and Instructions for Use

KS/PBNSC002	Pencil Brush, One Knot, Blue Nylon, Pointed, Screw - Hard
KS/BBRA001	Pencil Brush, One Knot, Black Bristle, RA
KS/BBRA002	Pencil Brush, One Knot, Black Bristle, Pointed, RA
KS/BBSC001	Pencil Brush, One Knot, Black Bristle, Screw
KS/BBSC002	Pencil Brush, One Knot, Black Bristle, Pointed, Screw
KSO/WN001	One Knot, White Nylon, Snap On
KSO/WN002	One Knot, White Nylon, Pointed, Snap On
MT/XWNRA	Microtuft, Extra White Nylon, RA
MT/PKNRA	Microtuft, Pink Nylon, RA - Soft
MT/PUNRA	Microtuft, Purple Nylon, RA - Medium
MT/PBNRA	Microtuft, Blue Nylon, RA - Hard
MT/XWNRA	Microtuft, Extra White Nylon, Screw
MT/PKNRA	Microtuft, Pink Nylon, Screw - Soft
MT/PUNRA	Microtuft, Purple Nylon, Screw - Medium
MT/PBNRA	Microtuft, Blue Nylon, Screw - Hard

RECOMMENDED SPEEDS

Junior Cup Brushes, Microtufts and Pencil Brushes One Knot

- Maximum speed 10,000 rpm
- Keep within the speed range of 1,500 – 10,000 rpm for the best work results and to increase the service life of the instrument.

Wheel Brushes RA

- Maximum speed 12,000 rpm
- Keep within the speed range of 1,500 – 12,000 rpm for the best work results and to increase the service life of the instrument.

INTENDED CLINICAL USE

These devices are indicated for use by qualified dental professionals for a wide range of procedures, including the cleaning and polishing of natural teeth, gold and amalgam fillings, composites, compomers and glass ionomer cements for removing plaque and stains. The devices may be used in combination with prophylaxis polishing pastes which are themselves CE marked medical devices intended for use in the oral cavity. No other materials, substances or gases are used within the device.

Please refer to the Sterilisation and Instructions for Use for Single Use Devices for the correct procedures.