

Sterilisation and Instructions for Use – Product Schedule

Mandrels

Classification: Class 2a Medical Device – (multi-use device)

GMDN Code	GMDN Term	Basic UDI-DI	Medical Device	MD Class
35170	Dental Mandrel	5051717MA01TG	Mandrel	Ila

TPR010	MANDREL SNAP-ON TO RA ADAPTOR
TPR010A	MANDREL SNAP-ON TO RA ADAPTOR
CTP054	MANDREL SC TO RA ADAPTOR
CTP055	MANDREL SNAP-ON TO RA ADAPTOR
CTP056	MANDREL SNAP-ON TO SC ADAPTOR
CTP061	POP-ON TO MANDREL - RA ADAPTOR
CTP062	SNAP-ON TO MANDREL - RA ADAPTOR

RECOMMENDED SPEEDS

- Never exceed the maximum permitted speed of the product mounted to the mandrel. The recommended speeds and maximum permitted speeds vary from product to product. Always check the recommended speed as stated on the packaging and the corresponding product schedule.
- If you exceed the maximum permitted speed, the polishers tend to vibrate, which can cause destruction to the shaft and/or make the instrument break, with a risk to the user, the patient or third parties.
- Keep within the speed range of the product being used for the best work results and to increase the service life of the instrument.
- Failure to comply with the maximum permitted speed produces an increased safety risk.

INTENDED CLINICAL USE

The devices are indicated for use by qualified dental professionals for holding prophylaxis brushes or polishers. The devices when used together, are designed and manufactured to perform safely for a wide range of procedures and are intended for use in the oral cavity.

Please refer to the Sterilisation and Instructions for Use for Multi-Use Devices for the correct procedures.