

Sterilisation and Instructions for Use – Product Schedule

Arkansas White Stones & Green Fine Stones Classification: Class 2a Medical Device – (multi-use device)

GMDN Code	GMDN Term	Basic UDI-DI	Medical Device	MD Class
31833	Dental Abrasive Point, reusable	5051717ABP1TW	Arkansas White Stones & Green Fine Stones	Ila

601/FG/3.0	Arkansas White Stones FG 3.0
601/RA/3.0	Arkansas White Stones RA 3.0
612/FG/3.0	Arkansas White Stones FG 3.0
612/RA/3.0	Arkansas White Stones RA 3.0
638/FG/3.0	Arkansas White Stones FG 3.0
638/RA/3.0	Arkansas White Stones RA 3.0
645/FG/3.0	Arkansas White Stones FG 3.0
645/RA/3.0	Arkansas White Stones RA 3.0
649/FG/2.7	Arkansas White Stones FG 2.7
649/RA/2.7	Arkansas White Stones RA 2.7
661/FG/3.0	Arkansas White Stones FG 3.0
661/RA/3.0	Arkansas White Stones RA 3.0
661F/RA/3.0	Green Fine Stones RA 3.0
601F/FG/3.0	Green Fine Stones FG 3.0
601F/RA/3.0	Green Fine Stones RA 3.0
612F/FG/3.0	Green Fine Stones FG 3.0
612F/RA/3.0	Green Fine Stones RA 3.0
638F/FG/3.0	Green Fine Stones FG 3.0
638F/RA/3.0	Green Fine Stones RA 3.0
645F/FG/3.0	Green Fine Stones FG 3.0
645F/RA/3.0	Green Fine Stones RA 3.0
649F/FG/2.7	Green Fine Stones FG 2.7
649F/RA/2.7	Green Fine Stones RA 2.7
661F/FG/3.0	Green Fine Stones FG 3.0

RECOMMENDED SPEEDS

- Never exceed the maximum permitted speed of the product. The recommended speeds and maximum permitted speeds vary from product to product. Always check the recommended speed as stated on the packaging and the corresponding product schedule.
- If you exceed the maximum permitted speed, the instrument tends to vibrate, which may cause damage to the shaft and/or make the instrument break, with a risk to the user, the patient or third parties. Failure to comply with the maximum permitted speed produces an increased safety risk.

INTENDED CLINICAL USE

The devices are indicated for use by qualified dental professionals for smooth grinding and polishing of filling materials, composite-filling materials, ceramic and enamel, porcelain and precious metals.

Please refer to the Sterilisation and Instructions for Use for Multi-Use Devices for the correct procedures.