

## Sterilisation and Instructions for Use – Product Schedule

### Prophy Polishing Brushes, RA and Screw

**Classification: Class 2a Medical Device – (single use device)**

GMDN Code	GMDN Term	Basic UDI-DI	Medical Device	MD Class
64015	Dental Professional Prophylaxis Brush, single use	5051717DP01TU	JC, Junior Cup Brushes	IIa
			KS, MT Brushes	

JC/WBRA	Junior Cup, White Bristle, RA
JC/WBSC	Junior Cup, White Bristle, Screw
JC/WNRA	Junior Cup, White Nylon, RA
JC/WNSC	Junior Cup White Nylon, Screw
JC/XWNRA	Junior Cup, Extra White Nylon, RA
JC/PKNRA	Junior Cup, Pink Nylon, RA
JC/PKNSC	Junior Cup, Pink Nylon, Screw
JC/PUNRA	Junior Cup, Purple Nylon, RA
JC/PUNSC	Junior Cup, Purple Nylon, Screw
JC/PBNRA	Junior Cup, Blue Nylon, RA
JC/PBNSC	Junior Cup, Blue Nylon, Screw
JC/SBNRA	Junior cup, Sky Blue Nylon, RA
JC/SBNSC	Junior Cup, Sky Blue Nylon, Screw
JC/GNRA	Junior Cup, Green Nylon, RA
JC/GNSC	Junior Cup, Green Nylon, Screw
JC/ORNRA	Junior Cup, Orange Nylon, RA
JC/YNRA	Junior Cup, Yellow Nylon, RA
JC/YNSC	Junior Cup, Yellow Nylon, Screw
JC/WNGRA	Junior Cup, White Nylon, Extra Soft (Gold Cup), RA
JC/WNGSC	Junior Cup, White Nylon, Extra Soft (Gold Cup), Screw
JP/WNRA	Junior Polymer Cup, White Nylon, RA
JP/PKNRA	Junior Polymer Cup, Pink Nylon, RA
JP/PUNRA	Junior Polymer Cup, Purple Nylon, RA
JP/PBNRA	Junior Polymer Cup, Blue Nylon RA
SET3026	Prophy Brush Set

KS/WBRA001	Pencil Brush, One Knot, White Bristle, RA
KS/WBRA002	Pencil Brush, One Knot White Bristle, Pointed, RA
KS/WBSC001	Pencil Brush, One Knot, White Bristle, Screw
KS/WBSC002	Pencil Brush, One Knot, White Bristle, Pointed, Screw
KS/WNRA001	Pencil Brush, One Knot, White Nylon, RA
KS/WNRA002	Pencil Brush, One Knot, White Nylon, Pointed, RA
KS/WNSC001	Pencil Brush, One Knot, White Nylon, Screw
KS/WNSC002	Pencil Brush, One Knot, White Nylon, Pointed, Screw
KS/XLWNRA001	Pencil Brush, Extra Large One Knot, White Nylon, RA
KS/PKNRA001	Pencil Brush, One Knot, Pink Nylon, RA
KS/PKNRA002	Pencil Brush, One Knot, Pink Nylon, Pointed, RA
KS/PUNRA001	Pencil Brush, One Knot, Purple Nylon, RA
KS/PUNRA002	Pencil Brush, One Knot, Purple Nylon, Pointed, RA
KS/PBNRA001	Pencil Brush, One Knot, Blue Nylon, RA
KS/PBNRA002	Pencil Brush, One Knot, Blue Nylon, Pointed, RA
KS/PKNSC001	Pencil Brush, One Knot, Pink Nylon, Screw
KS/PKNSC002	Pencil Brush, One Knot, Pink Nylon, Pointed, Screw
KS/PUNSC001	Pencil Brush, One Knot, Purple Nylon, Screw
KS/PUNSC002	Pencil Brush, One Knot, Purple Nylon, Pointed, Screw
KS/PBNSC001	Pencil Brush, One Knot, Blue Nylon, Screw
KS/PBNSC002	Pencil Brush, One Knot, Blue Nylon, Pointed, Screw

## Sterilisation and Instructions for Use

KS/ORNRA002	Pencil Brush, One Knot, Orange Nylon, Pointed, RA
KS/YNRA001	Pencil Brush, One Knot, Yellow Nylon, RA
KS/YNRA002	Pencil Brush, One Knot, Yellow Nylon, Pointed, RA
KS/YNSC001	Pencil Brush, One Knot, Yellow Nylon, Screw
KS/BBRA001	Pencil Brush, One Knot, Black Bristle, RA
KS/BBRA002	Pencil Brush, One Knot, Black Bristle, Pointed, RA
KS/BBSC001	Pencil Brush, One Knot, Black Bristle, Screw
KS/BBSC002	Pencil Brush, One Knot, Black Bristle, Pointed, Screw
KS/BNSC001	Pencil Brush, One Knot, Black Nylon, Flat trim, Screw

MT/XWNRA	Microtuft, Extra White Nylon, RA
MT/XWNSC	Microtuft, Extra White Nylon, Screw
MT/PKNRA	Microtuft, Pink Nylon, RA
MT/PKNSC	Microtuft, Pink Nylon, Screw
MT/PUNRA	Microtuft, Purple Nylon, RA
MT/PUNSC	Microtuft, Purple Nylon, Screw
MT/PBNRA	Microtuft, Blue Nylon, RA
MT/PBNSC	Microtuft, Blue Nylon, Screw
MT/SBNRA	Microtuft, Sky Blue Nylon, RA
MT/SBNSC	Microtuft, Sky Blue Nylon, Screw
MT/GNRA	Microtuft, Green Nylon, RA
MT/GNSC	Microtuft, Green Nylon, Screw
MT/YNRA	Microtuft, Yellow Nylon, RA

### RECOMMENDED SPEEDS

#### Junior Cup Brushes, Microtufts and Pencil Brushes One Knot

- Maximum speed 10,000 rpm
- Keep within the speed range of 1,500 – 10,000 rpm for the best work results and to increase the service life of the instrument.

### INTENDED CLINICAL USE

The devices are intended to be used by or under the instruction of a qualified dental professional on patients of any age for the prophylaxis cleaning or polishing of natural teeth, gold and amalgam filled surfaces in the mouth to remove plaque and stains. The devices may be used in combination with prophylaxis polishing pastes which are themselves CE marked medical devices intended for use in the oral cavity. No other materials, substances or gases are used within the device.

The devices are single use devices dental rotary instruments and are supplied mechanically clean but are not sterile and, therefore, should be sterilised before use.

**Please refer to the Sterilisation and Instructions for Use for Single Use Devices for the correct procedures.**