

Sterilisation and Instructions for Use – Product Schedule

Product: Prophy Cups, RA, Screw and Snap-On
Classification: Class 2a Medical Device – (single use device)

GMDN Code	GMDN Term	Basic UDI-DI	Medical Device	MD Class
46938	Dental Polishing Cup, single use	5051717DP01TU	Prophy Polishers	Ila

PRA001	Prophy Cup, Pink, 6 Web, Ring, RA
PRA002	Prophy Cup, Purple, 6 Web, Ring, RA
PRA003	Prophy Cup, Blue, 6 Web, Ring, RA
PSRA001	Prophy Cup, Pink, 6 Web, RA
PSRA002	Prophy Cup, Purple, 6 Web, RA
PSRA003	Prophy Cup, Blue, 6 Web, RA
PSRA004	Prophy Cup, White, 6 Web, RA
PSRA005	Prophy Cup, Yellow, 6 Web, RA
PSRA011	Prophy Cup, Pink, 4 Web, Ribbed, RA
PSRA012	Prophy Cup, Purple, 4 Web, Ribbed, RA
PSRA013	Prophy Cup, Blue, 4 Web, Ribbed, RA
PSRA014	Prophy Cup, Blue, 4 Web, Ribbed, RA
PSRA015	Prophy Cup, Yellow, 4 Web, RA
PSRA017	Prophy Cup, Green, 4 Web, RA
TPY002G	Prophy Cup, White, 4 Web, RA
EPR003	Prophy Cup, Mint, 6 Web Ring, RA
EPY002	Prophy Cup, Mint, 4 Web, Ribbed, RA
EPY003A	Prophy Cup, Mint, 6 Web, RA
PSC001	Prophy Cup, Pink, 6 Web Screw
PSC002	Prophy Cup, Purple, 6 Web Screw
PSC003	Prophy Cup, Blue, 6 Web Screw
PSC004	Prophy Cup, White, 6 Web Screw
PSC005	Prophy Cup, Yellow, 6 Web Screw
PSC011	Prophy Cup, Pink, 4 Web, Ribbed Screw
PSC012	Prophy Cup, Purple, 4 Web, Ribbed Screw
PSC013	Prophy Cup, Blue, 4 Web, Ribbed Screw
PSC014	Prophy Cup, Blue, 4 Web, Ribbed Screw
PSC015XH	Prophy Cup, Yellow, 4 Web, Ribbed Screw
PSC018	Prophy Cup, Green, 4 Web, Ribbed, Screw
EPR002A	Prophy Cup, White, 6 Web Screw
PSO001	Prophy Cup, Pink, 6 Web, Snap-On
PSO002	Prophy Cup, Purple, 6 Web, Snap-On
PSO003	Prophy Cup, Blue, 6 Web, Snap-On
PSO004	Prophy Cup, White, 6 Web, Snap-On
PSO005	Prophy Cup, Yellow, 6 Web, Snap-On
PSO011	Prophy Cup, Pink, 4 Web, Ribbed, Snap-On
PSO012	Prophy Cup, Purple, 4 Web, Ribbed, Snap-On
PSO013	Prophy Cup, Blue, 4 Web, Ribbed, Snap-On
PSO015	Prophy Cup, Yellow, 4 Web, Ribbed, Snap-On
PSO019	Prophy Cup, Green, 4 Web, Ribbed, Snap-On
PSO020	Prophy Cup, Yellow, 4 Web, Ribbed, Snap-On
PSO021	Prophy Cup, Grey, 4 Web, Ribbed, Snap-On
PSO022	Prophy Cup, Green, 4 Web, Ribbed, Snap-On
PSO023	Prophy Cup, White, 4 Web, Ribbed, Snap-On
PSO024	Prophy Cup, Blue, 4 Web, Ribbed, Snap-On
PSO028	Prophy Cup, Blue, 4 Web, Ribbed, Snap-On
PSO029	Prophy Cup, Blue, 4 Web, Ribbed, Snap-On

Sterilisation and Instructions for Use

EPR004A	Prophy Cup, White, 6 Web, Snap-On
EPR004B	Prophy Cup, White, 4 Web, Ribbed, Snap-On
PSMRA001	Prophy Cup Mini, Pink, 4 Web, Ribbed, RA
PSMRA002	Prophy Cup Mini, Purple, 4 Web, Ribbed, RA
PSMRA003	Prophy Cup Mini, Blue, 4 Web, Ribbed, RA
PSMSC001	Prophy Cup Mini, Pink, 4 Web, Ribbed, Screw
PSMSC002	Prophy Cup Mini, Purple, 4 Web, Ribbed, Screw
PSMSC003	Prophy Cup Mini, Blue, 4 Web, Ribbed, Screw
EPY004	Prophy Cup, Grey, Ribbed, RA
EPY005	Prophy Cup, Grey, Ribbed, Screw
TPR002A	Prophy Cup, White, 6 Web, Screw
TPR002B	Prophy Cup, White, 4 Web, Screw
TPR002H	Prophy Cup, Green, 4 Web Screw
TPR003	Prophy Cup, White, 6 Web, Ring, RA
TPR003B	Prophy Cup, Blue, 4 Web, RA
TPR003E	Prophy Cup, White, Ribbed, RA
TPR003G	Prophy Cup, Green, 4 Web, RA
TPR022	Prophy Cup Screw 6 Web Green
TPR023	Prophy Cup, Yellow, 4 Web, Ribbed, RA
TPR024	Prophy Cup, Grey, 4 Web, Ribbed, RA
TPY000	Prophy Cup, Grey, Ribbed, RA
TPY002	Prophy Cup, Mint, 4 Web, Ribbed, RA
TPY002A	Prophy Cup, Grey, 4 Web, RA
TPY003A	Prophy Cup, White, 6 Web, RA
SET3027	Prophy Polisher Set

RECOMMENDED SPEEDS

- Maximum speed 10,000 rpm
- Keep within the speed range of 1,500 – 10,000 rpm for the best work results and to increase the service life of the instrument.

INTENDED CLINICAL USE

The devices are intended to be used by or under the instruction of a qualified dental professional on patients of any age for the prophylaxis cleaning or polishing of natural teeth, gold and amalgam filled surfaces in the mouth to remove plaque and stains. The devices may be used in combination with prophylaxis polishing pastes which are themselves CE marked medical devices intended for use in the oral cavity. No other materials, substances or gases are used within the device.

Snap-On brushes and polishers are to be used with mandrel Ref: TPR010 or equivalent. Stoddard mandrels and brushes or polishers are proven to function together correctly by historical use with no adverse reports.

The devices are single use devices dental rotary instruments and are supplied mechanically clean but are not sterile and, therefore, should be sterilised before use.

Please refer to the Sterilisation and Instructions for Use for Single Use Devices for the correct procedures.